

To  
The Director  
SMD  
Chennai15-04-08  
Chennai

Lyphocheck Coagulation Control Lot 78152 and 78151 were tested on SMD- Fibrin 20 for PT and aPTT assays at SM Diagnostics' Lab and the following are the observations

Instrument : Fibrin 20

Assay: Prothrombin Time(PT)

Reagent : Tulip - Liquiplastin Lot No: A301713

Lyphocheck - Coagulation Control	Level 1 : Lot No 78151		Level 1 : Lot No 78152	
	Seconds	INR	Seconds	INR
Mean	10.85	0.76	18.68	1.72
Std Dev	1.29	0.13	1.52	0.21
No Of Points	30	30	30	30

Instrument : Fibrin 20

Assay: Activated Partial Thromboplastin Time(APTT)

Reagent : Tulip - Liquicellin Lot No: A303712

Lyphocheck - Coagulation Control	Level 1 : Lot No 78151	Level 1 : Lot No 78152
	Seconds	Seconds
Mean	27.16	40.66
Std Dev	7.79	18.59
No Of Points	30	30

Thank You  
Yours truly,  
For Bio-Rad Laboratories (I) Pvt.LtdBijesh.K.V   
Product Specialist – QSD

To  
The Director  
SM Diagnostics  
Chennai

3-2-08  
Chennai

**Dear Sir,**

**Please find attached the details as below:**

Lyphocek Coagulation Control Lot 78142 and 78143 were tested on SMD- Fibran 20 Software Version 1 & 2 on 2<sup>nd</sup> Feb 2008 at SM Diagnostics' Lab and the following are the observations

Assay : aPTT

Reagent Kit : TULIP Liquicelin-E Lot No: A303710 Exp: Sep 2008

Quality Control kit: Bio-Rad Lyphocek Coagulation Control Level 2 (Lot 78142)  
and Level 3 ( Lot 78143) Exp: 05/2009

Coagulo Meter : SMD Fibran 20 Software Version 2.0

Run No	Lyphocek Coagulation Lot 78142	Lyphocek Coagulation Lot 78143
1	49.79 Sec	62.70 Sec
2	42.79 Sec	66.79 Sec
3	48.59 Sec	64.79 Sec

Assay : aPTT

Reagent Kit : TULIP Liquicelin-E Lot No: A303710 Exp: Sep 2008

Quality Control kit: Bio-Rad Lyphocek Coagulation Control Level 2 (Lot 78142)  
and Level 3 ( Lot 78143) Exp: 05/2009

Coagulo Meter : SMD Fibran 20 Software Version 1.0

Run No	Lyphocek Coagulation Lot 78143
1	60.00 Sec
2	71.79 Sec
3	69.79 Sec

Thank You  
Yours truly,

For Bio-Rad Laboratories (I) Pvt.Ltd

  
Bijesh K. V.  
Product Specialist - QSD

